

# APAPRO®

Remineralizing Treatment Paste for Professional Use

Containing nano<mHAP>※

※SANGI's specially formulated nano Medical Hydroxyapatite , developed in 1980

High concentration nano hydroxyapatite enamel-restorative paste, for both professional use and patient home-care at the dentist's recommendation.



SANGI since 1980

## ■ Purpose of use

APAPRO enamel-restorative, hydroxyapatite-containing professional treatment paste replaces mineral lost from the enamel surface (microscopic surface fissures and subsurface lesions) during clinical procedures such as mechanical tooth cleaning (PMTC), bleaching and orthodontic work.

For use with a rubber cup or similar gentle polishing implement or in a retainer tray

- after PMTC
- after scaling
- after bleaching
- after removal of orthodontic brackets
- for general protective use

## ■ Storage

Replace cap firmly after use and store the product in a cool, clean place.

## ■ Precautions when using

- 1) Before use, confirm that there is no damage to the outer surface of the package or tube.
- 2) In case the patient shows any signs of soft tissue irritation after using this product, discontinue use.
- 3) As this product is a dental paste, please make sure it is not swallowed.

APPEN-1(N)



SANGI Co.,Ltd.

3-11-6 TSUKIJI CHUO-KU TOKYO 104-8440 JAPAN Tel: +81-3-3545 6013

✉ info@sangi-co.com

URL [www.sangi-co.com/en](http://www.sangi-co.com/en)

# APAPRO<sup>®</sup>

## Remineralizing Treatment Paste for Professional Use

Containing nano<mHAP><sup>※</sup>

### ■ How to use

#### 1. Application by rubber cup

- 1 Squeeze the paste onto a tray.
- 2 Apply rubber cup to a dental handpiece.
- 3 Scoop a small amount of paste into the rubber cup (before rotation).
- 4 Spread paste over the tooth surface (also before rotation).  
One scoop for each tooth is better than smoothing the paste over a wide range at the one time.
- 5 Rotate rubber cup and gently rub paste over the entire tooth, focusing especially on areas where plaque and stains can easily adhere (cervical and proximal surfaces).  
The rotation speed should be 500 to 750 rpm.  
Spend 5 to 10 seconds for each tooth, rotating the rubber cup slowly



POINT  
1

Remove plaque and stains completely, to ensure that the tooth surface is clean before treatment.



POINT  
2



For general use, apply 1-2g (roughly a 1-2cm strip) of APAPRO each to the upper and lower dentition.



POINT  
3

Move the cup gently and without applying pressure, as if placing a suction cup on the teeth.  
\* The cup is very soft and the paste will spread easily on the teeth.

#### 2. Application in a retainer tray

- 1 For home use, provide the patient with a customized tray and a tube of APAPRO Remineralizing Treatment Paste, with instructions for its periodic use, ideally for 5 minutes at a time after tooth brushing before bed at night.
- 2 Patient should spread 1-2g of APAPRO in the customized tray and fit the tray firmly over his or her teeth for 5 minutes, afterwards rinsing the mouth lightly with water.

